



The Implant Card is provided in paper format by Mt Ortho to all healthcare facilities, together with the medical device in question. The same card is available and downloadable from the company website www.mtortho.com on the homepage under the section "IMPLANT CARD".

2. INSTRUCTIONS FOR COMPILATION

2.1. PRE-PRINTED PAPER FORMAT

Medical personnel must provide the patient with a fully completed implant card, supplied in paper format together with the medical device.

Below is an example of the *Implant Card* document, including instructions for its completion:



Mt Ortho

IMPLANT CARD

 ?	<div style="background-color: #f0f0f0; width: 50px; height: 50px; margin: 0 auto; border: 1px solid #ccc; border-radius: 50%; display: flex; align-items: center; justify-content: center; font-weight: bold; font-size: 24px;">1</div>
<div style="border: 1px solid #ccc; padding: 2px; display: inline-block;">31</div>	<div style="background-color: #f0f0f0; width: 50px; height: 50px; margin: 0 auto; border: 1px solid #ccc; border-radius: 50%; display: flex; align-items: center; justify-content: center; font-weight: bold; font-size: 24px;">2</div>
 +	<div style="background-color: #f0f0f0; width: 50px; height: 50px; margin: 0 auto; border: 1px solid #ccc; border-radius: 50%; display: flex; align-items: center; justify-content: center; font-weight: bold; font-size: 24px;">3</div>


www.mtortho.com

1. Enter the patient's identification (ID);
2. Enter the implant date;
3. Enter the name of the hospital/healthcare facility where the procedure is performed.
4. The device type is indicated on the label;
5. Affix the label to the implant card.

2.2. COMPANY WEBSITE

To download the Implant Card from the company website, go to www.mtortho.com, select "IMPLANT CARD" on the homepage, then download and print the implant card.

Complete the implant card as described in the previous section and give it to the patient.

3. IMPORTANT INFORMATION

NB - If multiple components are implanted during a surgical procedure, an Implant Card must be provided for each implanted component.

It is specified below that healthcare institutions must provide patients with warnings, precautions, or measures to be taken by the patient or a healthcare provider regarding the following:

- mutual interference with reasonably foreseeable external influences (e.g., MRI), medical tests, or environmental conditions;
- information on the expected useful life of the device;
- information on any necessary follow-up;
- any other information necessary to ensure safe use of the device.